


Assembly and Maintenance Instructions FACESHIELD

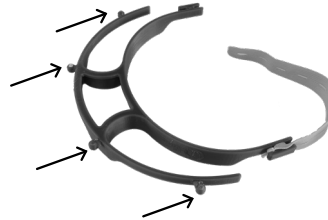

The BRUXSAFOL Faceshield is available in two different visor lengths:

Visor length 235 mm, art. no. HFS1752401S

Visor length 295 mm, art. no. HFS1753001S

Visor width 300 mm

Product Advantages:

- Very high transparency
- Anti-static (anti-fog) coating
- Frame made of high quality PP (food safe)
- Light and stable construction
- Head frame intended for long-term use (reusable)
- Frame, visors easy to clean and temperature stable
- Protective masks and glasses can easily be worn under the visor
- Made in Germany, ISO 9001 certified

Description:

The BRUXSAFOL Faceshield can assist in preventing the spread of infectious agents through droplets - for example sneezing, coughing and/or saliva. Our specially designed head frame provides sufficient spacing between the visor and face, allowing the comfortable use of glasses, safety goggles, and protective masks. The Faceshield is a supplementary apparatus and for the highest level of protection should be worn in combination with a breathing mask and safety goggles.

Thanks to the adjustable elastic band, the visor can be easily adjusted to fit nearly any head size.

Contents:

- Polypropylene head frame
- Highly transparent PET visor
- Adjustable elastic band
- Assembly instructions

Assembly:

Click the visor into the four brackets on the head frame (arrows in the picture, some pressure required) and hang the elastic band on the two ends, if necessary adjust the elastic band for a comfortable fit. You can find a video of the construction at www.bruxsafol.de/en/faceshield



Warnings:

The BRUXSAFOL Faceshield is not a medical product. No protection standard can currently be proven. The Faceshield is not sterile upon delivery.

The visor is not suitable for the following purposes:

- For protection against mechanical impacts
- For protection against burning, decomposing or chemical substances such as acids, light sources, gases or vapors of any kind
- For protection against sparks, radiation or ultraviolet rays
- When smoking or doing anything with or near flames or heat-emitting devices

Since this is a hygiene article, this product cannot be exchanged!

Care / Cleaning:

It is recommended to clean the face visor regularly inside and outside with disinfectant or a soapy solution (at least 20 seconds) using a soft cloth (a microfiber cloth is recommended in order to prevent scratches).

The head frame can be treated with disinfectant or disinfected using the steam sterilization process.